



## COVID-19 emergency face shields Assembly Instructions

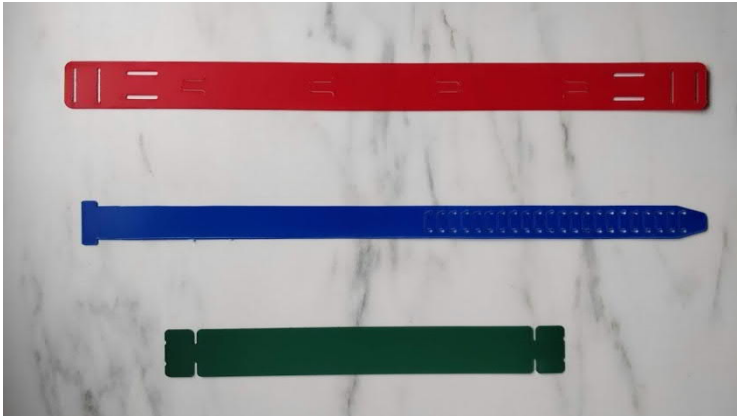

Each mask comes in four sections. The acetate face shield (not pictured), the face shield support (red), the head strap (blue) and the forehead strap (green).

Tuck one ear of the green forehead strap into the horizontal holes in the red support strap. Ensure the green strap does not cover the two holes at the end of the strap.

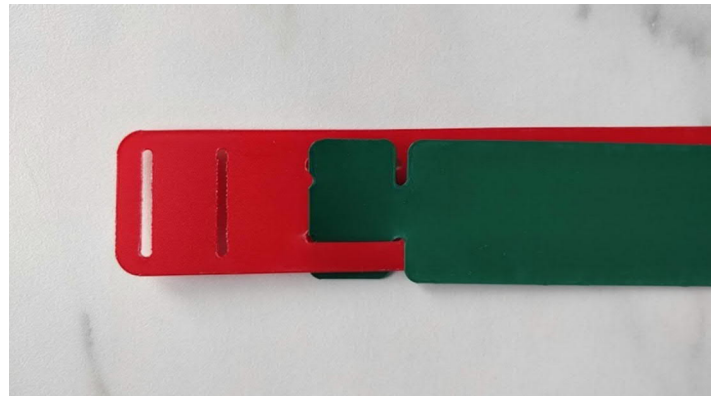

Tuck the 2<sup>nd</sup> ear into the 2<sup>nd</sup> horizontal hole in the red support strap.

Repeat on the opposite side.



Feed the blue headband strap through the first vertical holes from the front of the strap.



Feed the blue headband strap through the 2<sup>nd</sup> vertical hole.



Feed the blue headband strap through the 2<sup>nd</sup> vertical hole on the opposite side.


Feed the blue headband strap through the 1<sup>st</sup> vertical hole on the opposite side.

Do not pull tight. Place on head and pull through until comfortable.

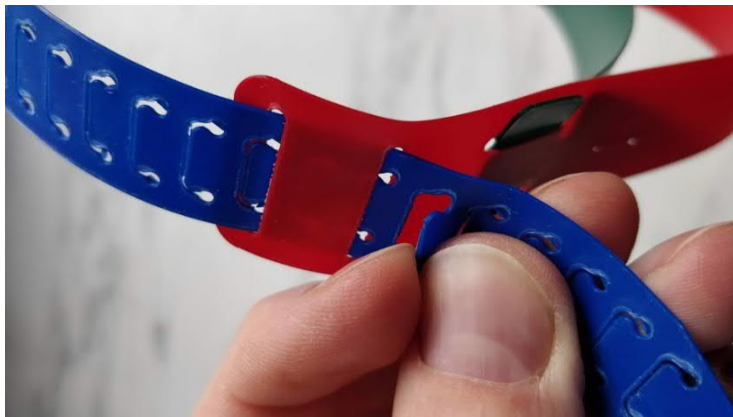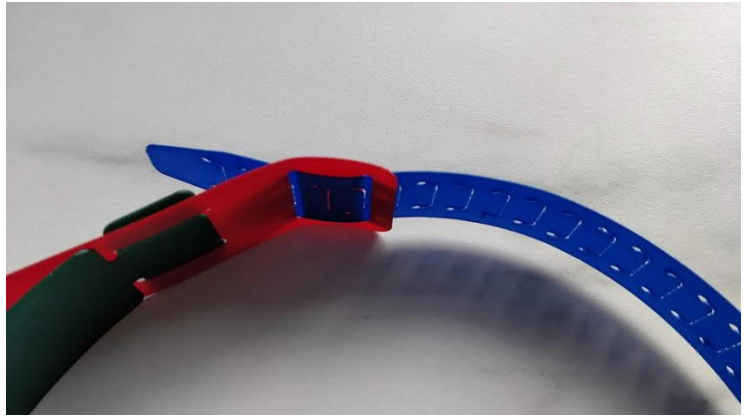

Lift the next tab up slightly with your nail and pull tight.

Once pulled tight, the tab will lock.

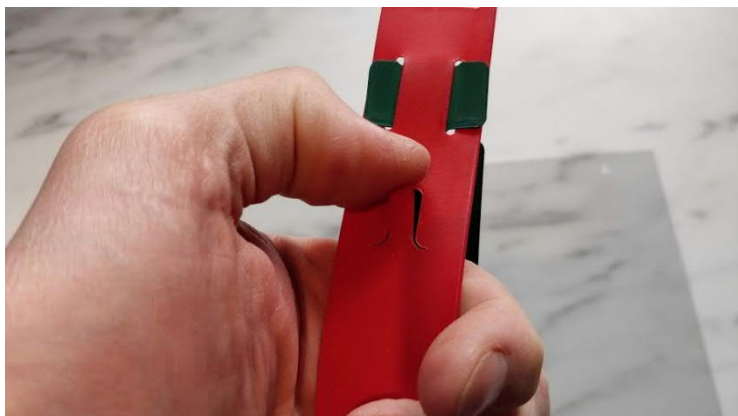

Lift the tabs on the red support strip.

Repeat for all four tabs



Hang acetate face shield onto tabs.  
You now have a functioning face shield!